

Codonics Safe Label System Hand Scanner Configuration

Release Notes

Overview

The Codonics Safe Label System™ Accessory Kit (Codonics part number SLS-ACC-KIT) includes a Symbol 9808 barcode scanner.

The Symbol barcode scanner is not configured as shipped. You must perform the procedure in “Configuring the Symbol 9808 Barcode Scanner” to configure it.

If you are using another type of barcode scanner, refer to “Parameters to Enable for Other Scanner Types” for a list of the parameters that must be enabled to properly configure the scanner for use with SLS.

Configuring the Symbol 9808 Barcode Scanner

Scan the 2D barcode below to configure the Symbol 9808 barcode scanner for any barcodes supported in the Administration Tool:



After successfully scanning this barcode, you will hear an audible beep, followed by an audible sound that indicates that the scanner is restarting.

Parameters to Enable for Other Scanner Types

The following parameters that must be enabled to properly configure a scanner for use with SLS.

Parameter	State
UPC-A	Enabled
UPC-E	Enabled
UPC-E1	Enabled
EAN-8/JAN 8	Enabled
EAN-13/JAN 13	Enabled
Transmit UPC-A Check Digit	Enabled
Transmit UPC-E Check Digit	Enabled
Transmit UPC-E1 Check Digit	Enabled
Code 128	Enabled
GS1-128 (formerly UCC/EAN-128)	Enabled
ISBT 128	Enabled
Check ISBT Table	Enabled
Code 39	Enabled
Convert Code 39 to Code 32 (Italian Pharmacy Code)	Enabled
Code 32 Prefix	Enabled
Interleaved 2 of 5 (ITF)	Enabled
GS1 DataBar-14	Enabled
GS1 DataBar Limited	Enabled
GS1 DataBar Expanded	Enabled
Data Matrix	Enabled
Inverse 1D	Inverse Autodetect
Data Matrix Inverse	Inverse Autodetect
Transmit Code ID	AIM Code ID
Presentation Performance Mode	Traditional

Technical Support

Please contact Codonics Technical Support at any time.

Phone: +1 (440) 243-1198
Email: support@codonics.com
Website: www.codonics.com

Get it all with just one call
1.800.444.1198

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice.

Copyright © 2010–2011 Codonics, Inc. Printed in the U.S.A. Part No. 901-249-002.03.



17991 Englewood Drive
Cleveland, OH 44130 USA
+1 (440) 243-1198
+1 (440) 243-1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Limited KK
New Shibaura Bldg. F1
1-3-11, Shibaura
Minato-ku, Tokyo, 105-0023 JAPAN
Phone: 81-3-5730-2297
Fax: 81-3-5730-2295