

Codonics SLS 500i[®]

Arm Mount Installation

CareFusion Pyxis[®] Anesthesia system

Overview

This document explains how to properly install the Codonics Extension Arm (Codonics part number SLS500-ARM-CF-EXT) and Spacer Kit (Codonics part number SLS500-ARM-CF-SP) accessories for the CareFusion Pyxis Anesthesia system Accessory Arm Mount (Codonics part number SLS500-ARM-CF).

The Extension Arm provides an additional four inches of vertical workspace under the Accessory Arm Mount and mounts to the existing mounting points on the Pyxis Anesthesia system 3500 or newer.

The Spacer Kit is for installation of the CareFusion Pyxis Anesthesia system Accessory Arm Mount with the handle accessory.

NOTE: These installation instructions assume the CareFusion Pyxis Anesthesia system Accessory Arm Mount has not been installed.

Tools Needed for All Installations

- ◆ 3/16-in. hex wrench

NOTE: Hex wrench provided with the CareFusion Pyxis Anesthesia system Accessory Arm Mount.

Parts List

The Codonics Extension Arm (Codonics part number SLS500-ARM-CF-EXT) includes the parts listed in the following table.

Part	Part No.	Qty.
Cap-head screw, 2-1/4-in. long	700-134-008	4
Cap-head screw, 1-3/4-in. long	700-134-006	4
Spacer, 1-in. long	830-048-005	4
Spacer, 3/8-in. long	830-048-004	4
Extension Arm	500-454-001	1

The Codonics Spacer Kit (Codonics part number SLS500-ARM-CF-SP) includes the parts listed in the following table.

Part	Part No.	Qty.
Cap-head screw, 1-3/4-in. long	700-134-006	5
Spacer, 1-in. long	830-048-005	4
Spacer, 3/8-in. long	830-048-004	4

Installation of the Extension Arm

If the Pyxis Anesthesia system handle accessory is not present, use the 1-in. long spacers and the 1-3/4-in. screws to mount the extension to the Pyxis Anesthesia system as shown in the following figure. For the top two screws, use the screws provided with the CareFusion Pyxis Anesthesia system Accessory Arm Mount.



If the handle accessory is present, use both the 1-in. long and 3/8-in. long spacers along with the 2-1/4-in. long screws to provide the necessary clearance.

Installation of the Spacer Kit

Use the Spacer Kit to provide the necessary clearance with the CareFusion Pyxis Anesthesia system handle accessory as shown in the figure below.



It is recommended to have the short spacer against the Accessory Arm Mount.

Technical Support

If problems occur when installing these accessories, contact Codonics Technical Support.

Phone: +1.440.243.1198
Email: support@codonics.com
Website: www.codonics.com

Get it all with just one call
800.444.1198

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice.

Copyright © 2014-2015 Codonics, Inc. Printed in the U.S.A. Part No. 901-269-002.04.



17991 Englewood Drive
Middleburg Heights, OH
44130 USA
+1.440.243.1198
+1.440.243.1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Trading Co, Ltd.
317 Xianxia Rd. Building B
Unit 1412
Changning Dist., Shanghai
P.R. China, 200051
86-21-62787701
86-21-62787719 Fax

Codonics Limited KK
AQUACITY 9F,
4-16-23, Shibaura
Minato-ku, Tokyo,
108-0023 JAPAN
81-3-5730-2297
81-3-5730-2295 Fax