

IMPORTANT NOTICE

Product Bulletin No. 409

This notice affects Virtua products running software prior to version v5.1.0.

Security patch update to prevent WannaCry malware vulnerability smb_4548_1.0

Virtua products with software prior to version v5.1.0 may be vulnerable to the WannaCry ransomware attack. To remove this vulnerability, a security patch can be applied. Once applied, the text string "smb_4548_1.0" will be appended to the version string. To check the version string, navigate to the Help Tab on the Virtua User Interface.

Software Version

Version: v4.2.0 / smb_4548_1.0

Example: Software version v4.2.0 with security patch applied.

The patch release is available as a free download from our website:

<http://www.codonics.com/Support/>

Codonics Technical Support

Phone: +1.440.243.1198
E-mail: support@codonics.com
Website: www.codonics.com